**THUASNE®**

# Ortel C1 Classic

<b>fr</b>	Collier cervical souple.....	5
<b>en</b>	Soft cervical collar .....	6
<b>de</b>	Weiche Halskrause .....	7
<b>nl</b>	Flexibele nekbrace .....	8
<b>it</b>	Collare cervicale morbido .....	10
<b>es</b>	Collarín cervical flexible .....	11
<b>pt</b>	Colar cervical flexível.....	12
<b>da</b>	Smidig nakkekrave.....	14
<b>fi</b>	Taipuisa kaulatuki .....	15
<b>sv</b>	Mjuk nackkrage .....	16
<b>el</b>	Αυχενικό κολλάρο μαλακό.....	17
<b>cs</b>	Pružný krční límec.....	19
<b>pl</b>	Kołnierz elastyczny do unieruchomienia szyi .....	20
<b>lv</b>	Mīksta kakla ortoze .....	21
<b>lt</b>	Lankstus kaklo įtvaras.....	22
<b>et</b>	Pehme emakakaela krae .....	24
<b>sl</b>	Opornica za vrat, prožna .....	25
<b>sk</b>	Ohybný krčný golier .....	26
<b>hu</b>	Lágy nyakmerevítő .....	28
<b>bg</b>	Мекa цервикална яка .....	29
<b>ro</b>	Guler cervical flexibil .....	30
<b>ru</b>	Мягкий шейный воротник.....	31
<b>hr</b>	Mekana ortoza za vratnu kralježnicu.....	33
<b>zh</b>	软质颈圈.....	34
<b>ar</b>	طوق للعنق مرن .....	37

Instructions d'utilisation - Instructions for use - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Инструкция по эксплуатации - Upute za uporabu - 使用指南 - إرشادات الاستعمال

	a	b			
0	24 - 29 cm	6 cm	-	-	-
1	28 - 33 cm		7,5 cm	9 cm	11 cm
2	34 - 39 cm				
3	40 - 46 cm	-			

<b>fr</b>	(a) Tour de cou (b) Hauteur de cou
<b>en</b>	(a) Neck circumference (b) Neck height
<b>de</b>	(a) Halsumfang (b) Halshöhe
<b>nl</b>	(a) Omtrek van de nek (b) Hoogte van de nek
<b>it</b>	(a) Circonferenza del collo (b) Altezza del collo
<b>es</b>	(a) Circunferencia del cuello (b) Altura del cuello
<b>pt</b>	(a) Perímetro do pescoço (b) Altura do pescoço
<b>da</b>	(a) Halsens omkreds (b) Halsens højde
<b>fi</b>	(a) Kaulanympärys (b) Kaulan korkeus
<b>sv</b>	(a) Runt halsen (b) I höjd med halsen
<b>el</b>	(a) Περίμετρος λαιμού (b) Ύψος λαιμού
<b>cs</b>	(a) Obvod krku (b) Výška krku
<b>pl</b>	(a) Obwód szyi (b) Wysokość szyi



<b>lv</b>	(a) Kakla apkārtmērs (b) Kakla augstums
<b>lt</b>	(a) Kaklo apimtis (b) Kaklo aukštis
<b>et</b>	(a) Kaela ümbermõõt (b) Kaela pikkus
<b>sl</b>	(a) Obseg vratu (b) Dolžina vratu
<b>sk</b>	(a) Obvod krku (b) Výška krku
<b>hu</b>	(a) Nyak kerülete (b) Nyak magassága
<b>bg</b>	(a) Размер на яката (b) Височина на яката
<b>ro</b>	(a) Circumferința gâtului (b) Înălțimea gâtului
<b>ru</b>	(a) Обхват шеи (b) Высота шеи
<b>hr</b>	(a) Opseg vrata (b) Visina vrata
<b>zh</b>	( a ) 颈围 ( b ) 颈高
<b>ar</b>	محيط العنق (a) ارتفاع العنق (b)

<b>fr</b>	Maintien	Effet proprioceptif
<b>en</b>	Support	Proprioceptive effect
<b>de</b>	Halt	Propriozeptive Wirkung
<b>nl</b>	Fixatie	Proprioceptieve werking
<b>it</b>	Sostegno	Effetto propioceettivo
<b>es</b>	Sujeción	Efecto propioceptivo
<b>pt</b>	Manutenção	Efeito propioceetivo
<b>da</b>	Støtte	Proprioceptisk effekt
<b>fi</b>	Tuki	Liikeaistiin kohdistuva vaikutus
<b>sv</b>	Stöd	Proprioceptiv effekt
<b>el</b>	Συγκράτηση	Αποτέλεσμα ιδιοδεκτικότητας
<b>cs</b>	Stabilizace	Proprioceptivní účinek
<b>pl</b>	Podtrzymywanie	Propriocepcja
<b>lv</b>	Atbalsts	Proprioceptīvs efekts
<b>lt</b>	Palaikymas	Propriocepcinis poveikis
<b>et</b>	Hooldus	Proprioceptiivne toime
<b>sl</b>	Oprijem	Proprioceptivni učinek
<b>sk</b>	Podpora	Proprioceptívny účinok
<b>hu</b>	Támasz	Proprioceptív hatás
<b>bg</b>	Поддръжка	Проприоцептивен ефект
<b>ro</b>	Susținere	Efect proprioceptiv
<b>ru</b>	Стабилизация	Проприоцептивный эффект
<b>hr</b>	Potpora	Učinak proprioceptivnog vježbanja
<b>zh</b>	稳定	本体感
<b>ar</b>	الدعم	مفعول استقبال الحس العميق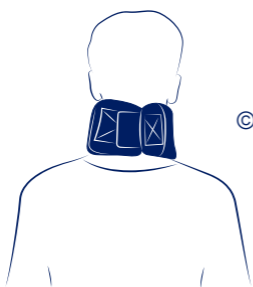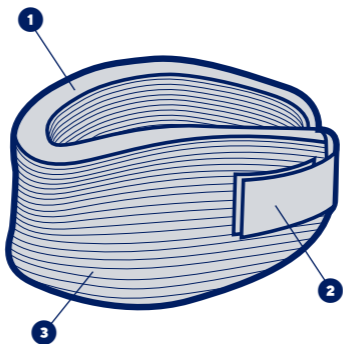

## COLLIER CERVICAL SOUPLE

### Description/Destination

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## **SOFT CERVICAL COLLAR**

### **Description/Destination**

Rounded edges for appropriate comfort. ①

Precise self-fastening adjustment. ②

Convenient removable cover. ③

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### **Composition**

Cotton - polyamide - polyester - polyurethane.

### **Properties/Mode of action**

Support.

Proprioceptive effect.

### **Indications**

Cervical arthritis.

Cervical syndrome.

Dropped head syndrome.

Mild whiplash/mild sprain.

Post-operative use.

Rehabilitation after wearing a rigid cervical collar.

Torticollis (stiff neck).

### **Contraindications**

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of traumatic injuries: severe sprain, fracture.

Do not use for prolonged wear without medical supervision.

Do not use the product if the diagnosis has not been confirmed.

### **Precautions**

Strictly comply with your healthcare professional's prescription and recommendations for use.

In the event of discomfort, significant hindrance, pain, abnormal sensations, remove the device and consult a healthcare professional.

Choose the appropriate size to fit the patient, referring to the size chart.

Verify the product's integrity before every use.

It is recommended that a healthcare professional supervises the first application.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device if it is damaged.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support without excessive compression.

Do not wear the device in a medical imaging machine.

Do not wear the device while sleeping.

Do not wear the product more than 10 days without your healthcare professional's recommendation.

### **Undesirable side-effect**

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

In the event of long-term wear of the product, possible secondary risks related to muscle atrophy.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### **Instructions for use/Application**

Open the self-fastener.

Place the collar around the neck ①.

Close the self-fastener at the nape of the neck ②.

Ensure the strap is not fastened too tight ③.

### **Care/Maintenance**

See product label. Close the self-fastening tabs before washing. Hand wash.

Do not dry clean. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not tumble-dry. Do not iron. Dry away from any direct heat source (radiator, sun, etc.). Dry flat.

### **Storage**

Store at room temperature, preferably in the original packaging.

### **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2028903 (2023-10)
**THUASNE UK Ltd**
Thuasne SAS - SIREN/RCS Nanterre 542 091 186