THUASNE®

Cicatrex

de Gebrauchsanweisung - Narbenkompressionsbekleidung
en Instructions for use - Scar compression garment
fr Consignes d'utilisation - Vêtement compressif de cicatrisation
it Istruzioni per l'uso - Indumenti di compressione per cicatrici
es Instrucciones de utilización - Prendas de compresión para cicatrices
pt Instruções de uso - Vestuário de compressão para cicatrizes
nl Gebruiksaanwijzing - Compressiekleiding voor littekens
da Brugsanvisning - Kompressionstøj til arbehandling
sv Bruksanvisning - Ärrkompressionsplagg
fi Käyttöohje - Painevaatteet arpihoitoon
no Bruksanvisning - Arrkompresjonsbekledning
el Οδηγίες χρήσης - Συμπιεστικό ένδυμα για ουλές
cs Návod k použití - Kompresivní prádlo pro terapii jizev
pl Sposób użytkowania - Odzież kompresyjna do blizn
et Kasutusjuhend - Kompressioonriietus armide raviks
lv Lietošanas instrukcija - Rētu kompresijas apģērbi
lt Naudojimo instrukcija - Kompresinė apranga randams gydyti
sk Návod na použitie - Kompresívne odevy na liečbu jaziev
sl Navodila za uporabo - Kompresijska oblačila za brazgotine
hu Használati utasítás - Hegterápiás kompressziós ruházat
ro Instrucțiuni de utilizare - Îmbrăcămintă compresivă pentru cicatrici
bg Инструкция за употреба - Компесионно облекло при белези
hr Upute za uporabu - Kompresijska odjeća za ožiljke
ru Инструкция по использованию - Изделия из компрессионного трикотажа для лечения рубцов
tr Kullanım talimatı - Yara izi kompresyon kıyafeti
ko 사용 지침 - 흉터 압박복
ja 使用説明書 - 瘢痕治療用圧着衣
ar تعليمات الاستخدام - ملابس ضاغطة للندوب

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Purpose

Compression textiles are basic elements in scar tissue therapy. The objective is to achieve flat and elastic scar tissue which does not reduce the quality of life. Cicatrex compression garment is used for the treatment of scars. It applies continuous planar pressure onto the scar area. It compensates for the natural resilience and surface elasticity of the skin. It prevents the scar from rising above the level of the skin. It reduces scar perfusion. It supports parallel arrangement of collagen fibres. It protects the fresh scar against external influences and alleviates itching. It prevents inflammatory reactions in the tissue. It improves the appearance and function of the scar. Compression therapy generally is a long-term treatment and requires the corresponding products, which – if the medical indication is given – the patient him/herself applies upon the doctor's instruction.

The Cicatrex compression garment can be combined with:

Silon-TEX®

Silon-TEX® is a textile carrier material with medical silicone layer applied. It creates a moist environment and keeps the scar soft, prevents the scar from drying out and positively supports ripening of the scar.

Pads

Pads increase the local pressure on the scar tissue. The increased pressure can prevent excessive growth of scar tissue (increased collagen production). The scar becomes more flat and soft. The pads can be integrated into the product by means of a pocket.

Silicone pads

Silicone pads are customised and can be used individually. They are placed on the body externally and reach their effect primarily through the properties of the material and pressure on the skin. Their purpose is to make the scar tissue more soft and elastic without drying it out.

The combination of continuous surface pressure of the compression garment and pad (pad, Silon-TEX® or separate silicone pads) promotes scar healing and improves the scar tissue visually as well as functionally.

Properties

Cicatrex compression garment combines efficient medical care and high wearing comfort. The material's high elasticity provides for ease of putting the material on or taking it off, and it also ensures great freedom of movement. Cicatrex is hypoallergenic, prevents inflammations in the tissue, relieves itching and provides comprehensive protection for the fresh scar.

Indications

Damage of the skin and the skin's healing function with hypertrophic scarring or keloid formation.

- Following injuries / burns / surgical interventions
- Skin grafts
- For improvement of scar / wound healing until complete healing of the scar
- For the stabilisation of surgical results
- Medical care of postoperative and chronic edema as well as lymphedema (in special cases)
- Prevention of chemotherapy-induced polyneuropathy (CIPN) on the distal extremities
- Pain with lipedema
- Vascular malformations: Venous, lymphatic and arteriovenous malformations

Absolute contraindications

Peripheral arterial occlusive disease (PAOD) stage III/IV; decompensated heart failure; septic phlebitis; phlegmasia coerulea dolens; acute erysipelas; acute bacterial, viral or allergic inflammations; untreated or advanced dermatoses / contact dermatitis; untreated open wounds; swelling of extremities of unknown origin; untreated malignant tumours.

Do not use Silon-TEX® in case of fissured skin or open wounds; also, do not use it for transplants which did not heal in a stable manner.

Relative contraindications

Pronounced exudative dermatoses; malignant lymph oedema; peripheral arterial occlusive disease (PAOD) stage I/II; peripheral neuropathy associated with diabetes mellitus; sensitivity disorders of extremities; compensated heart failure; skin infections; dermatoses (e.g. blistering dermatoses); intolerance of pressure or of product ingredients; primary chronic polyarthritis. In case of open wounds in the treatment area, the wound area must first be covered with an adequate wound dressing or a bandage before the compression garment is applied.

Thuasne Deutschland GmbH accepts no liability for failure to observe these contraindications.

Side effects

No side effects are known if properly used. In case of any negative changes during the application, please consult your doctor or the medical retailer immediately. In case of complaints in connection with the product, such as damage of the knitted fabric or lack of fit, please contact directly your medical retailer.

Please note the following if Silon-TEX® is sewn into your compression garment: Should negative changes

occur, such as itching or a skin allergy due to silicone intolerance, please contact immediately your doctor or your medical retailer. Should incompatibilities regarding one or several ingredients of this product be known, please consult your doctor prior to use. If skin irritations or allergies persist, stop using Silon-TEX®. The manufacturer shall not be liable for damages / injuries which resulted due to improper handling or misuse.

Precautions

Follow the instructions given by your doctor, by an orthopaedic technician or medical expert personnel. If you experience severe pain or discomfort, please see your doctor, an orthopaedic technician or medical expert personnel. Store at room temperature, preferably in the original packaging. To guarantee its effectiveness and for reasons of hygiene, the product may not be passed on to other patients. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Directions for putting the garment on

Your medical retailer will show you the correct application. Ideally, you should wear the compression garment 24 hours a day. Protect the fine knitted fabric against damage from rough fingernails or jewellery. Please use rubber gloves when getting dressed, or use suitable dressing aids if there are difficulties. Protect the skin against sunlight by applying sunscreen under the clothing. Pay particular attention to protecting the scar tissue. If Silon-TEX® is sewn in or a separate silicone pad is used, please observe the following: Thoroughly clean the scar and surrounding skin. Keep the scar tissue dry and grease-free. Do not use ointments, creams, gels or lotions with Silon-TEX® or a silicone pad. Place the Silon-TEX® or silicone pad directly and smoothly onto the scar area. Do not use the product if there are skin irritations. Do not use it with known silicone intolerance. Discontinue the application in case of skin reactions and consult a doctor. Do not use Silon-TEX® or silicone pads on open wounds.

Care

Compression garments should be washed daily, if possible. The garment fabric not only collects remainders of ointments and grease but also skin scales, sweat and dust. Regular washing will protect you from infections and your garment from wear. You need a spare set of garments so that you need not interrupt your compression therapy while the other set is washed and dried.

Always turn your compression garment inside out to wash it. Use a mild detergent for washing by hand or for machine wash (gentle cycle at 40° C) without optical brightener and fabric softener. In a laundry bag, compression garments are specially protected. In order to reduce the drying period, put the compression garments into a terry cloth towel, roll it up firmly and press the garment out (do not wring it out). Do not leave the moist compression garment in the wet towel. Hang up your garment loosely and leave it to air-dry. Never put it in the sun or onto a radiator to dry.

Do not expose compression garments to any damaging effects, such as: strong sunlight, intense heat, ointments, oils and greases, stain remover, gasoline and aggressive detergents. When using a separate silicone pad, please clean this daily with lukewarm water to remove any residue, it is not suitable for cleaning in the washing machine.

Material composition

Please find the exact information on the sewn-in fabric label.

Note: The sewn-in fabric label is an important component of the compression garment. If you remove this labelling from a Thuasne® product, any claim for warranty, repair or replacement shall be waived.

Material composition of Silon-TEX®: Silicone layer: 100 % medical grade silicone, carrier material: 81 % polyamide, 19 % elasthan

Storage information and duration of use

Store in a dry place and protect from sunlight. For series products, a maximum shelf life of 36 months applies. This is followed by the duration of use of 6 months for the compression garment. The hourglass pictogram shows the duration of use printed on the carton label. Customised manufacture concerns products for immediate use.

For serial sizes and custom-tailored garments, we guarantee their medical efficacy for a maximum period of wear of six months. Prerequisite for this will be the proper treatment of the compression materials (for example, their care or putting them on or taking them off). If necessary, your doctor should then prescribe new compression garments. In case of any significant change of relevant body sizes (e.g. due to a successful therapy, weight change), new measurements and a new supply may also be justified earlier already.

Disposal

You may dispose of your medical compression garment via the residual or household waste. No special criteria exist for their elimination.

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Thuasne Cervitex Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
Thuasne Scandinavia AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd