# THUASNE SPORT

- fr Coudière anti-épicondylite Thuasne Sport
- en Thuasne Sport anti-epicondylitis elbow brace
- de Ellenbogenstütze gegen Epicondylitis  
Thuasne Sport
- nl Thuasne Sport anti-epicondylitis elleboogbrace
- it Gomitiera anti-epicondilitite Thuasne Sport
- es Codera antiepicondilitis Thuasne Sport
- pt Cotoveleira antiepicondilitite Thuasne Sport
- da Albuebind mod epicondylitis Thuasne Sport
- fi Thuasne Sport -epikondyliittituki
- sv Thuasne Sport armbågsskydd mot epikondylit
- el Επίδεσμος αγκώνα κατά της επικονδυλίτιδας  
Thuasne Sport
- cs Loketní bandáž proti epikondylitidě Thuasne Sport
- pl Opaska zapobiegająca entezopatii stawu  
łokciowego (łokieć tenisisty) Thuasne Sport
- lv Elkoņa balsts epikondilīta novēršanai  
Thuasne Sport
- lt Alkūnės įtvarymas epikondilitui gydyti  
"Thuasne Sport"
- et Epikondüliidivastane küünarnukutugi  
Thuasne Sport
- sl Komolčna opornica proti epikondilitisu  
Thuasne Sport
- sk Podpora lakťa proti epikondylitíde Thuasne Sport
- hu Teniszkönyök elleni könyökrögzítő Thuasne Sport
- bg Налакътник срещу епикондилит Thuasne Sport
- ro Cotieră antiepicondilită Thuasne Sport
- ru Бандаж на локоть при эпикондилите  
Thuasne Sport
- hr Steznik za epikondilitis Thuasne Sport
- zh Thuasne Sport 抗肱骨外上髁炎护肘
- ar داعم مرفق مضاد لالتهاب اللقيمة Thuasne Sport

	<b>S</b>	<b>M</b>	<b>L</b>	<b>XL</b>
8 cm	22 - 24	25 - 27	28 - 30	31 - 33

<b>fr</b>	Tour de l'avant-bras
<b>en</b>	Forearm circumference
<b>de</b>	Umfang des Unterarms
<b>nl</b>	Omtrek van de voorarm
<b>it</b>	Circonferenza dell'avambraccio
<b>es</b>	Circunferencia del antebrazo
<b>pt</b>	Perímetro do antebraço
<b>da</b>	Omkreds af underarm
<b>fi</b>	Yläkäsväaren ympärysmitta
<b>sv</b>	Runt underarmen
<b>el</b>	Περίμετρος του αντιβραχίου
<b>cs</b>	Pro předloktí
<b>pl</b>	Opaska na przedramię
<b>lv</b>	Apkārt apakšdelmam
<b>lt</b>	Dilbio apimtis
<b>et</b>	Küünarvarre ümbermõõt
<b>sl</b>	Obseg podlakti
<b>sk</b>	Obvod predlaktia
<b>hu</b>	Alkar kerülete
<b>bg</b>	Обиколка на предмишницата
<b>ro</b>	Circumferința antebrățului
<b>ru</b>	Обхват предплечья
<b>hr</b>	Opseg podlaktice
<b>zh</b>	前臂周长
<b>ar</b>	محيط الساعد

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---

## **en**

# **THUASNE SPORT ANTI-EPICONDYLITIS ELBOW BRACE**

### **Description/Destination:**

This device is an elbow brace providing joint support and stabilisation.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

### **Composition:**

Textile components: neoprene - polyamide - elastane.

Strap: polyester - polyoxymethylene.

Insert: silicone.

### **Properties/Mode of action:**

Even compression on the joint.

Improved proprioception.

Rapid perspiration wicking (perforated fabric).

Removable tightening strap.

### **Indications:**

Treatment of lateral or medial epicondylitis (tennis/golfer's elbow).

Prevention of elbow tendinopathy recurrence.

Support for physical activity resumption.

### **Contraindications:**

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

### **Precautions:**

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

### **Undesirable side-effect:**

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

### **Instructions for use/Application:**

Slip on the elbow brace, taking care to position the inserts on the forearm.

Fasten the strap on the elbow brace using the touch fastener.

Do not tighten excessively to avoid any tourniquet effect.

### **Care:**

Product can be washed in accordance with the conditions shown on this notice and on the label. Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.).

### **Storage:**

Store at room temperature, preferably in the original packaging.

### **Disposal:**

Dispose of in accordance with local regulations.

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©Thuasne - 2025601 (2021-04)
Thuasne SAS - SIREN/RCS Nanterre 542 091 186