
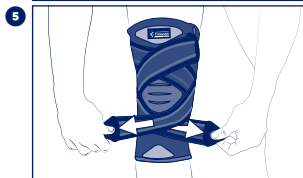
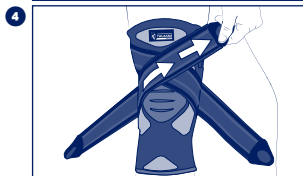
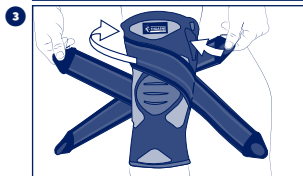
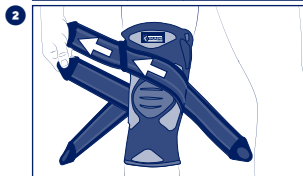
**THUASNE****SPORT**

<b>fr</b>	<b>Genouillère strapping Novelastic Thuasne Sport .....</b>	<b>4</b>
<b>en</b>	<b>Thuasne Sport Novelastic strapping knee brace.....</b>	<b>5</b>
<b>de</b>	<b>Novelastic Thuasne Sport-Strapping-Kniebandage... </b>	<b>7</b>
<b>nl</b>	<b>Thuasne Sport Novelastic kniebrace met elastische banden.....</b>	<b>9</b>
<b>it</b>	<b> Ginocchiera strapping Novelastic Thuasne Sport.....</b>	<b>11</b>
<b>es</b>	<b>Rodillera strapping Novelastic Thuasne Sport .....</b>	<b>13</b>
<b>pt</b>	<b>Joelheira de strapping Novelastic Thuasne Sport .....</b>	<b>15</b>
<b>da</b>	<b>Knæbind med rem Novelastic Thuasne Sport.....</b>	<b>16</b>
<b>fi</b>	<b>Nauhallinen Novelastic Thuasne Sport -polvituki .....</b>	<b>18</b>
<b>sv</b>	<b>Knästöd med elastiskt band Novelastic Thuasne Sport .....</b>	<b>20</b>
<b>el</b>	<b>Επιγονατίδα strapping Novelastic Thuasne Sport.....</b>	<b>22</b>
<b>cs</b>	<b>Kolenní ortéza strapovací Novelastic Thuasne Sport....</b>	<b>24</b>
<b>pl</b>	<b>Bandażowa orteza kolanowa Novelastic Thuasne Sport .....</b>	<b>25</b>
<b>lv</b>	<b>Elastīga ceļa ortoze Novelastic Thuasne Sport .....</b>	<b>27</b>
<b>lt</b>	<b>Apjuosiamas kelio įtvaras „Novelastic Thuasne Sport“ .....</b>	<b>29</b>
<b>et</b>	<b>Novelastic Thuasne Sport fikseerivate rihmadega põlvekaitse.....</b>	<b>31</b>
<b>sl</b>	<b>Opornica iz trakov za koleno Novelastic Thuasne Sport .....</b>	<b>32</b>
<b>sk</b>	<b>Kolenná ortéza s popruhmi Novelastic Thuasne Sport... </b>	<b>34</b>
<b>hu</b>	<b>Thuasne Sport Novelastic pántos térdmervítő.....</b>	<b>36</b>
<b>bg</b>	<b>Пристягаща на коленка Novelastic Thuasne Sport..</b>	<b>37</b>
<b>ro</b>	<b>Genunchieră cu benzi ajustabile Novelastic Thuasne Sport .....</b>	<b>39</b>
<b>ru</b>	<b>Ременный наколенник Novelastic Thuasne Sport... </b>	<b>41</b>
<b>hr</b>	<b>Steznik za koljeno s trakama Novelastic Thuasne Sport... </b>	<b>43</b>
<b>zh</b>	<b>Thuasne Sport Novelastic肌内效护膝 .....</b>	<b>45</b>
<b>ar</b>	<b>Novelastic Thuasne Sport داعم ركبة مزود بشرائط الشد .....</b>	<b>47</b>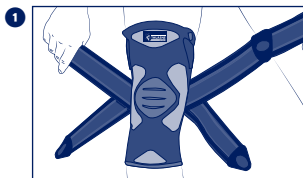

 cm	<b>S</b> 32 - 34	<b>M</b> 35 - 37	<b>L</b> 38 - 41	<b>XL</b> 42 - 45	<b>XXL</b> 46 - 49
---	------------------	------------------	------------------	-------------------	--------------------

<b>fr</b>	Circonférence de genou en légère flexion
<b>en</b>	Knee circumference in slight flexion
<b>de</b>	Knieumfang bei leichter Beugung
<b>nl</b>	Omvang van de licht geplooid knie
<b>it</b>	Circonferenza del ginocchio in leggera flessione
<b>es</b>	Circunferencia de la rodilla con ligera flexión
<b>pt</b>	Circunferência do Joelho em ligeira flexão
<b>da</b>	Knæets omkreds i let bøining
<b>fi</b>	Polven ympärys hieman koukistettuna
<b>sv</b>	Omkrets runt knäet i lätt böjning
<b>el</b>	Περιφέρεια γόνατος σε ελαφρά κάμψη
<b>cs</b>	Obvod kolene v mírném ohybu
<b>pl</b>	Obwód kolana przy lekkim zgięciu
<b>lv</b>	Celgala apkārtmērs nelielā saliekumā
<b>lt</b>	Šiek tiek sulenкто kelio apimtis
<b>et</b>	Kergelt pinguldutult põlve ümber
<b>sl</b>	Obseg kolena pri rahlem upogibu
<b>sk</b>	Obvod kolena s miernym ohybom
<b>hu</b>	Térd kerülete enyhe hajlításban
<b>bg</b>	Обиколка на коляното при лека флексия
<b>ro</b>	Circumferința genunchiului în flexiune ușoară
<b>ru</b>	Обхват колена в состоянии небольшого сгибания
<b>hr</b>	Obuhvaća koljeno u blagoj fleksiji
<b>zh</b>	轻度弯曲的膝关节周长
<b>ar</b>	محيط الركبة في حالة النسي الخفيف

## fr GENOUILLÈRE STRAPPING NOVELASTIC THUASNE SPORT

### Description/Destination

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## en THUASNE SPORT NOVELASTIC STRAPPING KNEE BRACE

### Description/purpose

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Bilateral product.

Available in 5 sizes.

The device is composed of:

- an anatomically-shaped knitting integrating a light and soft knit at the back of the knee,

- flexible lateral reinforcement,
- a double functional strapping ensuring good lateral support,
- anti-slip system to prevent the knee brace slipping.

#### **Composition**

Textile components: polyamide - polyester - polyurethane - elastan - elastodiene - silicone.

Rigid components: polyamide - steel - polyethylene.

#### **Properties/Mode of action**

Stabilisation of the knee ligaments provided by the double strapping system.

An anatomically-shaped knitting for even compression and proprioception.

#### **Indications**

Joint instability/laxity.

Knee pain and/or swelling management.

Sport activities resumption.

#### **Contraindications**

Do not use in the event of known allergy to any of the components.

Do not apply the product in direct contact with broken skin.

History of venous or lymphatic disorders.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

#### **Precautions**

Verify the product's integrity before every use.

Do not use the product if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

For hygiene and performance reasons, do not re-use the product for another patient.

Consult a healthcare professional in case of uncertain diagnosis.

It is recommended to adequately tighten the device to achieve support/immobilisation without restricting blood circulation.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

In case of doubt before any sports activity, consult a healthcare professional.

Do not wear the product in a medical imaging machine.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

#### **Undesirable side-effects**

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents related to the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

#### **Instructions for use/Application**

It is recommended to wear the device directly on the skin, unless contraindicated.

To avoid damage to the knit fabric, please make sure that the self-gripping pads are always fastened to the designated self-gripping surfaces when putting it on and taking it off.

Slip the knee brace on the leg with your foot extended, making sure that the patella is properly positioned in the flexible knitted zone provided for this purpose. Thuasne Sport logo must be on the upper part. Pull and fix the crossover straps by respecting the fitting sequence detailed in diagrams 1, 2, 3, 4 and 5.

Each strap and piece has 1, 2 or 3 colour dots, which show how to fix the end to each corresponding grip base.

Start by fixing the end that has 1 colour dot to the grip patch with 1 colour dot.

The same for the end with 2 colour dots which is attached to the grip patch with 2 colour dots and so on...

For comfort during wear, make sure the straps are not creased. If you feel the need to tighten the straps during an activity, make sure you tighten them following the steps indicated in the fitting instructions.

#### **Care**

Close the self-fastening tabs before washing. Machine washable at 30 °C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Squeeze out excess water. Dry away from any direct heat source (radiator, sun, etc.). If the device is exposed to seawater or chlorinated water, make sure to rinse it in clear water and dry it.

#### **Storage**

Store at room temperature, preferably in the original packaging.

#### **Disposal**

Dispose of in accordance with local regulations.

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©Thuasne - 2034801 (2022-08)
**THUASNE UK Ltd**
Thuasne SAS – SIREN/RCS Nanterre 542 091 186