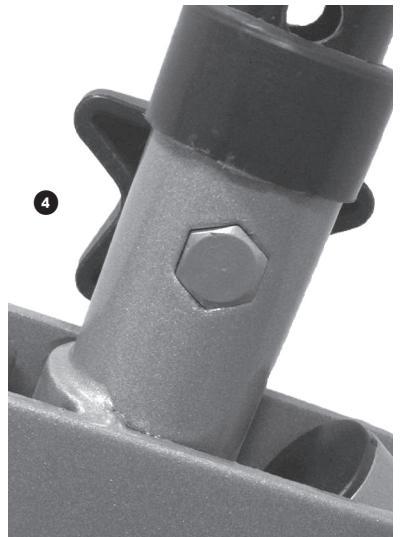
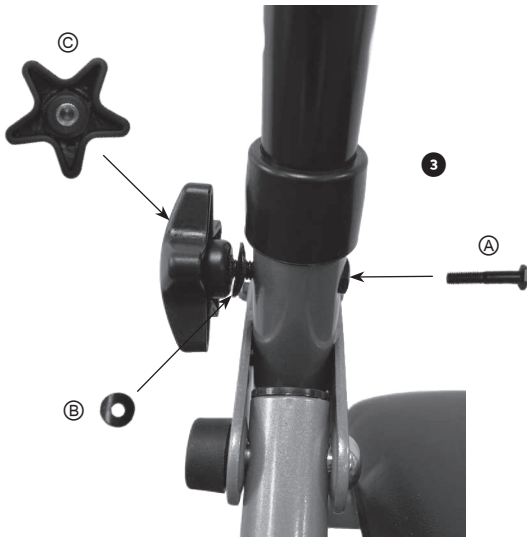
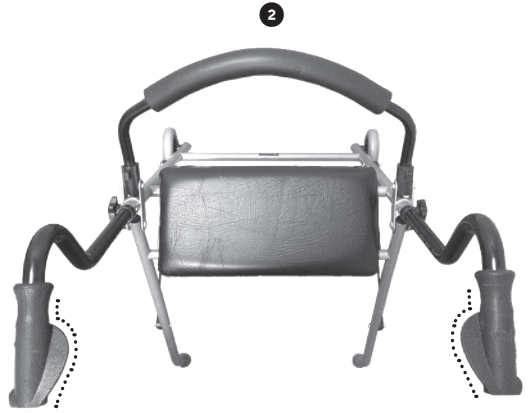
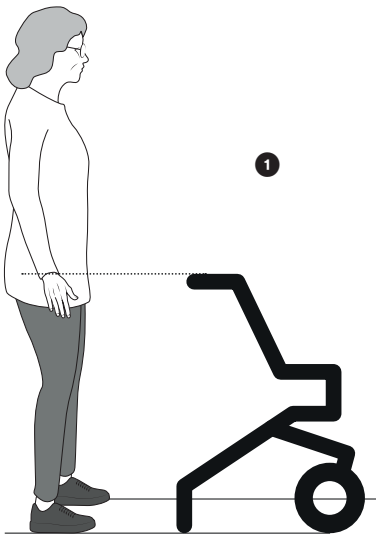
**THUASNE**

# Duo Confort

V2220

fr	Rollateur .....	4
en	Rollator .....	5
de	Rollator .....	7
nl	Rollator .....	8
it	Rollator .....	10
es	Andador .....	11
cs	Chodítka .....	13
pl	Podpórka .....	14
sk	Štvorkolka .....	16

5

6

**MD**

<b>fr</b>	Dispositif médical
<b>en</b>	Medical device
<b>de</b>	Medizinprodukt
<b>nl</b>	Medisch hulpmiddel
<b>it</b>	Dispositivo medico
<b>es</b>	Producto sanitario
<b>cs</b>	Zdravotnický prostředek
<b>pl</b>	Wyrób medyczny
<b>sk</b>	Zdravotnícka pomôcka

<b>fr</b>	Lire attentivement la notice
<b>en</b>	Read the instruction leaflet carefully
<b>de</b>	Lesen Sie die Gebrauchsanweisung sorgfältig durch
<b>nl</b>	Lees voor gebruik de gebruiksaanwijzing aandachtig door
<b>it</b>	Leggere attentamente le istruzioni
<b>es</b>	Leer atentamente las instrucciones
<b>cs</b>	Pozorně si přečtete návod
<b>pl</b>	Należy dokładnie przeczytać instrukcję
<b>sk</b>	Tento návod si pozorne prečítajte

# ROLLATEUR DUO CONFORT

## Description/Destination

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### Description/Destination

2-wheel rollator.

This device helps temporarily or permanently mobility impaired individuals to maintain their freedom of movement by providing them with an additional point of support. It also has a seat with a backrest for taking short rests. For indoor use only.

### Main components

Frame: steel

Handles: PVC

Seat (cover): PVC

Backrest (cover): thermoplastic

Wheels/tires: PVC

Ferrules: synthetic rubber

### Indications

Post-operative and post-traumatic support.

Muscle weakness, neuromuscular disorders, stability problems, impaired balance, arthritis of the lower limbs.

Degenerative conditions of the lower limbs.

### Contraindications

Not suitable for patients weighing > 100 kg.

Do not use in the event of known allergy to any of the components.

Do not use in the event of severe perceptual function deficit or severe balance disorders.

### Precautions

#### Important warnings

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Before using this product, check that:

- it is stable and rigid,
- the wheels are in good condition and turn freely,
- no components are incorrectly attached or not properly screwed in place.

The health care professional who supplied the product must make sure that it is appropriate for the patient.

It is recommended that a health care professional assembles and adjusts the device and supervises the first use.

Follow the advice of the health care professional who prescribed or supplied the product and consult them in case of doubt.

Do not use the device for any use other than its intended use: risk of injury or damage to equipment.

Never allow children to play with the product.

Beware of the risks of pinching when handling the device.

Always completely unfold the rollator before use.

Do not make any mechanical changes to the device, this could compromise its safety. Only original spare parts and accessories have to be used.

Take care when using the device on slippery surfaces (wet floors or wet tiles, etc.), uneven or sloping surfaces, or unstable surfaces (gravel).

Do not use on stairs or escalators.

Do not leave exposed to direct sunlight for a long period of time. Check the temperature of the device surfaces, particularly before sitting down on the seat: risk of burns.

### Undesirable side-effect

- Risk of discomfort in the event of incorrect adjustment of the device.
- Danger of falling or loss of balance, which can lead to serious injuries.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

**Instructions for use**

**Assembly/unfolding**

1. Opening the rollator

Push the central black bar located under the seat downwards until the rollator is fully unfolded, then put the seat down.

2. Assembling the handles/adjusting the height

The height of the device must be adjusted with reference to the height of the patient. Incorrect adjustment can impair performance and place too much strain on the user.

The top of the handle must be approximately at the height of the wrist fold when the user's arms are alongside the body, wearing usual shoes. ①

- Insert the handles into the tubes of the frame. Be careful to ensure that the right and left handles are installed the correct way round. ②

- Slide them to the desired height.

- Place the bolt (A) in the hexagonal opening, then put the washer (B) in place before tightening the knob (C). ③ Bolt A must be fully pushed in. ④

Make sure that the handles are fixed at the same height on both sides.

3. Installing the backrest

Assemble the backrest by pressing its push-buttons while inserting the tubes into the plastic supports.

Take care to ensure the backrest is the correct way up. ⑤

Check that the push-buttons are properly engaged in the holes in the plastic supports.

**Folding**

- To fold away the rollator for easier transport, place one hand on one handle and lift up the seat with the other hand, then pull on the belt to raise the black bar ⑥.

**Important warnings**

Before using the seat, check that the rollator is on a flat, stable surface.

Do not sit at an angle or lean to the side or back.

The rollator seat is only intended for taking short rests.

Beware of the risks of pinching when handling the device.

Do not attach handbags, carrier bags or any objects other than those supplied with the device: risk of tipping.

The device must not be used as an aid to get up from the sitting position: risk of unbalancing.

**Maintenance and care**

**Maintenance**

If any part of the device becomes detached, loose or visibly damaged, contact a specialized dealer or health care professional.

If necessary, replacement parts and original accessories are available from your Thuasne product dealer.

**Care**

Clean with a mild cleaning agent and a soft cloth.

Do not clean the handles, the wheels or the ferrules with greasy substances: risk of slipping.

If necessary, use a mild contact disinfectant or 70° alcohol. In an institution, follow the recommendations of the hygiene manager.

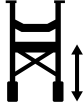
**Storage**

Store at room temperature, in a dry, ventilated place.

**Disposal**

Disposal of in accordance with local regulations.

**Technical characteristics**

<p>Handle adjustment height</p>  <p>79 - 96 cm Steps of 2.8 cm</p>	<p>Total width</p>  <p>58.5 cm</p>
<p>Total depth</p>  <p>65 cm</p>	<p>Total weight with/without backrest</p>  <p>5.7/5.3 kg</p>
<p>Seat depth</p>  <p>22 cm</p>	<p>Seat height</p>  <p>52 cm</p>
<p>Seat width</p>  <p>42.5 cm</p>	<p>Maximum user weight</p>  <p>100 kg max</p>

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©Thuasne - 2027001 (2021-02)